



## UR235010 - UR275030

### Leg Bags

T-Tap with Elastic Straps

### INSTRUCTION FOR USE

#### DESCRIPTION

m|devices Leg Bags are designed to be unobtrusive offering discreet storage under clothing.

The various volume capacity sizes available are 350mL, 500mL and 750mL with three optional tube lengths of 10cm and 30cm with bonded stepped connector, cater for all user requirements.

T-Tap, non-return valves and elastic straps are inclusive in all Leg Bag codes.

**THE BELOW IS ONLY A SUGGESTION AND FACILITY PROTOCOL AND OR EDUCATION PROTOCOL MUST BE FOLLOWED FOR ALL CLINICAL PROCEDURES WHERE THIS PRODUCT IS USED.**

#### CAUTION

- DO NOT use if the package or product has been damaged or contaminated.
- DO NOT store at extreme temperatures and humidity, avoid direct sunlight.

#### STEPS

1. Remove the blue protective cap from the stepped connector on the inlet drainage tube and using aseptic technique to connect directly into the funnel end of the Foley Catheter. Ensure the connection is secure.
2. Using the elastic straps, apply the Leg Bag to the desired thigh by wrapping the two straps on the bag around the thigh and secure at desired position using the white button. Push the white button through to the corresponding button-hole on the opposing strap until secure.
3. To empty the bag (the T-Tap will be in a closed position when removed from the packaging); hold the T-Tap mechanism in your hand, place your thumb on the white capped button and depress until it sits firmly against the T piece. Urine will flow out.
4. Once empty, using the same mechanism to open the T-Tap, depress the blue capped button until it sits firmly against the T piece.

#### PRECAUTIONS

- Ensure the Leg Bag is never raised higher than the level of the bladder.
- Ensure the Leg Bag tubing is free from kinks at all times when connected to the catheter.
- Ensure aseptic technique is maintained when handling connecting and disconnecting the Leg Bag from the urinary catheter or other drainage bags.
- Ensure connections are always secure between the Foley Catheter and drainage outlet tube.

**m|devices**

27 Llewellyn Avenue  
Villawood NSW 2163  
Australia

T +61(0)2 8718 2800

F +61(0)2 8718 2801

**m|devices<sup>®</sup>**