




Product Specification Sheet

Product Code:	KWSSLBR
Product Description:	KW/MASTR SOLUBL SEAM LAUNDRY BAG RED
Product Information:	LDPE bag with a water soluble tie attached on the bag, with a water soluble tie for closing.
Carton Barcode:	19325191025242

Physical Specification

Length (A):	990mm
Width (B):	720mm
Gussett (C):	500mm
Open width:	720mm
Thickness:	30 microns
Material:	LDPE
Virgin or Recycled:	Virgin
Film Colour:	Red
Printed:	No
Bag Weight:	41 Grams

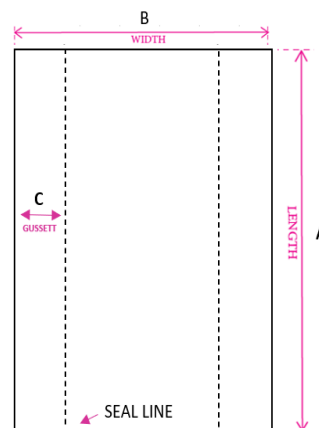

Quality Specification

Tensile load MD:	14.7 MPA
Tensile load TD:	20.1 MPA
	7.5 MPA/MD
Base seal strength:	10.0 MPA/TD
Dart impact:	20gram /30 thickness film
Lift capacities:	5KG

Packing Specification

Count per pack:	50pcs
Flat pack or Roll:	Flat pack
Pack per outer bag:	5 packs
Core material:	No core
Outer bags per carton:	5 plastic bags
Carton dimensions:	400x300x170mm
Weight per carton:	10 KG
Cartons per layer:	in loose pack
Layers per Pallet:	in loose pack

Country of Origin: China

Created: 30/03/2021

Last Edited: 30/03/2021

Head Office:

Bunzl Australasia Limited
2/700 Springvale Road, Mulgrave Victoria 3170
Ph:(+61) 3 9590 300


<https://shop.bunzl.com.au>

Disclaimer:

The information provided is based on the product's manufacturer and a copy of manufacturer's declaration is held on file. We have every reason to believe that the information provided is accurate and will be valid until revoked.



Carton layout:
(Image)

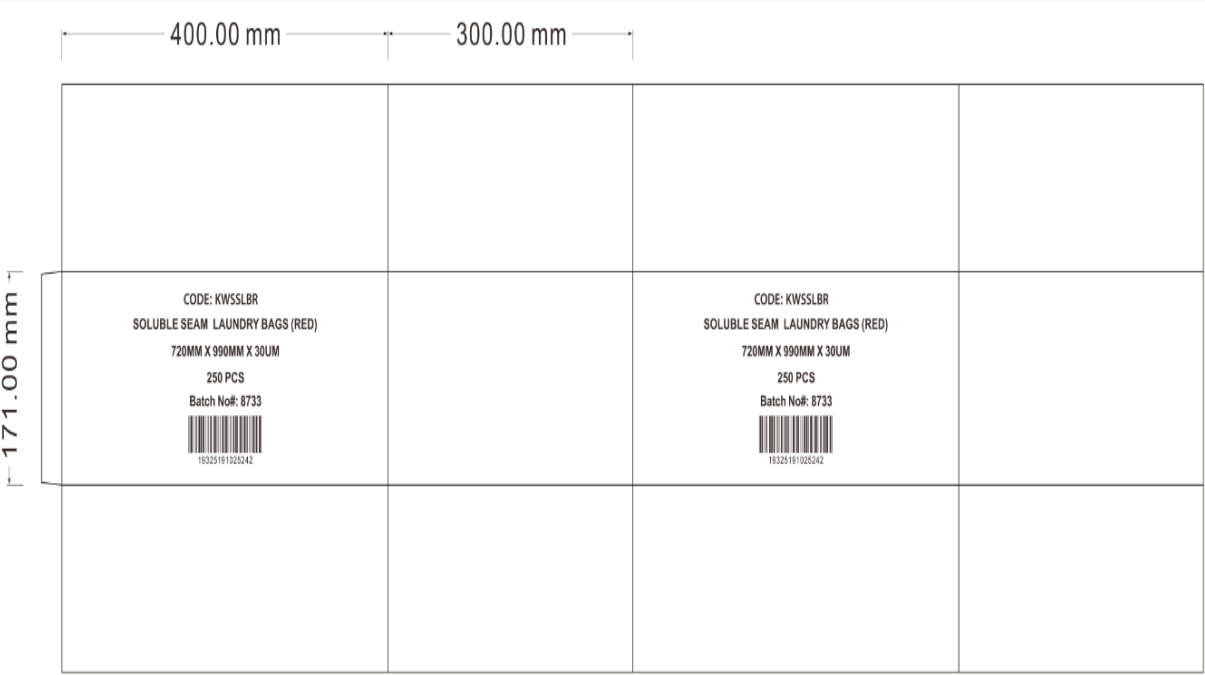

Outer bag layout:

(Image)



SOLUBLE SEAM LAUNDRY BAGS OPERATING INSTRUCTION

For the safe containment and transportation of soiled or infectious linen

The soluble seam bag fulfills the need to effectively and conveniently treat contaminated linen. The soluble seam bag prevents the handling of contaminated linen from its initial pick-up to completion of its wash.

Features & Benefits

Construction

The soluble seam bag is manufactured from polyethylene with a dissolvable seam fixed down the middle of the bag. When the seam dissolves during the initial stages of the laundry cycle, the bag opens up allowing the infectious and/or soiled linen to be released. A detachable soluble tie tape is also used to seal the bags opening.

Suitability

The opening allows linen to be placed into the bag with ease. The seam is located on the outside of the bag to prevent seam dissolving upon contact with the damp linen and during transportation

.

Infection control


The soluble seam bag liner confines the infectious linen inside the bag and prevents bacteria or infection transmission.

Sizes

Standard 720mm wide x 990mm depth x 30um Box 250 (packed into 5 packets of 50)

Standard is available in RED or YELLOW

Instructions for use:

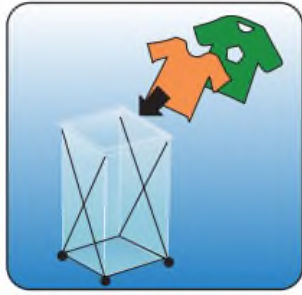

STEP 1

Place soiled items into the Soluble Seam Laundry Bag.



STEP 2

Dispel the air from the bag and tie the top of the bag with the dissolvable tie provided. **DO NOT TIE A KNOT WITH THE BAG.**




STEP 3

Place the bag into the washing machine and select the wash program.



STEP 4

Within 30 seconds the bag's seam will dissolve and the contents will begin to wash.

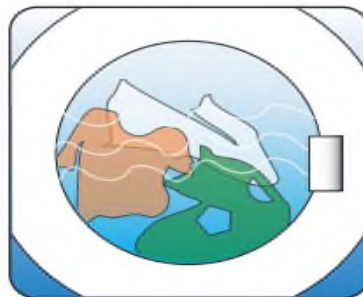

STEP 5

At the end of the wash, remove the used bag and discard to plastic recycle bin or general waste.

Comes standard with a soluble tie to seal the bag. This helps to eliminate the transfer of viruses and bacteria via the air and keeps items secure ready for washing. If excessively wet, items are placed into the bag, use a towel or similar to wrap the items as excessive moisture in the bag will start to dissolve the seam before the bag is delivered to the laundry causing the bag to prematurely open.