



Item number 1000025110

ABENA®

ABENA Slip Basic, M3, white, blue color code, Basic

- ✓ Textile-like backsheet: comfortable and breathable
- ✓ Top-Dry System for dry surface to protect the skin
- ✓ Efficient core technology for fast absorption
- ✓ Curved leg elastics for optimum fit.
- ✓ Leakage barriers for extra protection
- ✓ Wetness indicator
- ✓ Odor Control



Product Description

Abena Slip Basic is our range of bladder protection briefs for moderate to heavy incontinence. The products feature the Top-Dry system for optimum comfort and skin integrity. The anatomically shaped briefs offer an optimal fit combined with curved leg elastics for extra comfort. All products are fitted with leakage barriers and a built-in cross barrier at rear for extra protection. The odor system minimizes unpleasant smells. The combi tapes ensure an optimum fit and easy repositioning.



Specifications

Brand	ABENA
Sub-brand	Slip Basic
Size	M3
Color	White
Features	Blue color code
Length/depth	840 mm
Width	670 mm
Waist circumference	70-110 cm
Tolerance	+/- 10%
Absorption Rothwell	2900 ml
Gender	Unisex
Quality level	Basic
Certifications	CE. MD. UKCA.
CE Class (Medical Devices)	Class I
Product or test standards	ISO 11948-1, ISO 14001, ISO 9001
Directives, regulations and acts	MDR (EU) 2017/745
Because of the products composition it has an expiry date from the production date on:	5 years
Storage Instructions	Store dry, room temperature and no direct sunlight.
Product Disposal Instructions	Can be disposed of with normal household waste sorted according to local regulations.
Packaging Disposal Instructions	Can be recycled or incinerated.

Instructions for use/application

Bladder protection briefs for moderate to heavy incontinence. . Product size recognition by color codes, names and numbering system. It is important to always measure the hip circumference to select the right size and to choose the correct absorption level to avoid leakages. Fold the brief

lengthways with the backsheet turned outwards to create a bowl shape. Pass the brief from front to back. Fan out the front of the brief, creating a channel shape between the legs. Fix the bottom tapes angled slightly upwards on both sides and then fix top tapes angled downwards for a secure and comfortable fit.

Packaging data

Unit	Contains	Length	Width	Height	EAN
cII	4 pck	57.6 cm	39.6 cm	36.7 cm	5703538498800
pck	24 pcs	38 cm	18 cm	26 cm	5703538498794



Regulation (EU) 2017/745 is a regulation of the European Union on the clinical investigation and sale of medical devices for human use. It repeals Directive 93/42/EEC, which concerns medical devices, and Directive 90/385/EEC, which concerns active implantable medical devices, on 26 May 2021.



The CE mark guarantees that a product is safe to use and complies with all safety precautions. CE stands for Conformité Européenne (European Conformity) and is mainly found on electronic equipment, safety equipment, construction products, and medical devices.



The UKCA mark guarantees that a product is safe to use and complies with all safety precautions in line with product standards and legislation of the UK.



Odor control is a critical factor in ensuring discreteness for the user. ABENA's incontinence products are designed and engineered to ensure that minimal Odor arises from the product.



Effective leakage barriers, both visible and invisible and latex-free elastics ensure a snug fit, which reduces the risk of leakages.



Wetness indicator is a liquid-sensitive strip that changes colour or fades out indicating when it is time to change the product.



The product does not contain latex.



Unique Top-Dry Technology that provides a rapid absorption and ensures a dry surface even after multiple voids.

