






Lohmann & Rauscher


Discover the new side of Debrisoft® – **Debrisoft® Duo.**

When I want
to improve, I

debrisoft it.



Two sides,
two textures,
one solution.



When simplicity matters, I
debrisoft it.



**Efficient 2-in-1
debridement pad:**

- the **original soft, white side** removes debris, exudate and biofilm from the wound
- the **new textured beige side** makes it easy to loosen firmly adherent, fibrinous slough*

Developed based on user experience to allow more effective debridement of firmly adhering, fibrinous slough.

Debrisoft® Duo would be recommended by 100% of users in an acceptance test.

Debrisoft® Duo
Simplify your debridement routine.



1. Moisten
Moisten both sides of Debrisoft Duo with water or saline before usage.



2. Adapt to needs
Start with either side of Debrisoft Duo, depending on the wound condition and switch as needed.



3. Debride
Applying gentle pressure, work the pad over the wound or the affected skin area for 2–4 minutes, until devitalised tissue and debris are removed.



4. Done
Dispose of Debrisoft products after use.

When I need rapid results, I
debrisoft it.



Debrisoft® Duo
10 x 10 cm

- designed for debridement and wound cleansing
- you have the flexibility to use either side of the pad depending on the wound condition and patient need
- the textured beige side makes it easy to loosen firmly adherent fibrinous slough
- the soft white side removes debris, exudate and biofilm from the wound

Debrisoft® Lolly

- for cleansing and debridement of deep wounds and hard-to-reach areas
- ideal for surgical invasive wounds




Debrisoft® Duo
13 x 20 cm

The best just got better –
designed based on user experience:

Highly effective

Absorbs, binds and removes debris, exudate and bacteria.
Removes 99% of biofilm*.

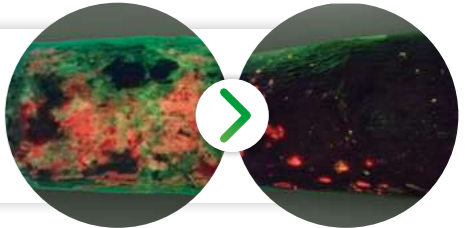

Source 1

**Easy and
safe to use**

High user satisfaction,
rapidly visible results
within 2–4 minutes.



Source 2

**Adapt to wound
situation and
patient needs**

Debrisoft Duo was developed to enable more effective debridement of firmly adhering, fibrinous slough.



*Based on in vitro data




The Debrisoft® product family

Three options – one solution.



Want to use the original?
Get helpful tips here ...

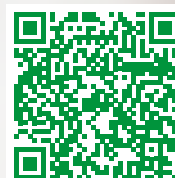

Experience Debrisoft® Duo for yourself.


Debrisoft® Duo
sterile, individually sealed

Size (cm)	REF	Boxes/Carton
10 x 10	159 990	5/50
13 x 20	159 991	5/50

Debrisoft® Pad
sterile, individually sealed

Size (cm)	REF	Boxes/Carton
10 x 10	34 321	5/50
13 x 20	34 323	5/50

Try Debrisoft® Duo now!
For more flexibility
in debridement.

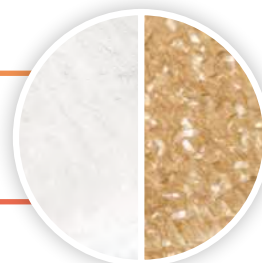

Debrisoft® Lolly
sterile, individually sealed

REF	Boxes/Carton
33 224	5/50

Please note the instructions for use.

Sources:

- 1) Images: Photos used with patient's kind consent, UK.
- 2) Images: Gabriele Stern, ATZ-Stern, Bocholt, Germany.

For any queries please contact your local Lohmann & Rauscher (L&R) Sales Representative or L&R Customer Service on:

AUS: 1300 572 869 | **NZ:** 0800 572 869 | **Email:** info@au.LRmed.com

Lohmann & Rauscher Pty Ltd · 321 Ferntree Gully Road · Mount Waverley, Victoria 3149 · Australia

Lohmann & Rauscher Limited · 21 Queen Street · Auckland 1010 · New Zealand